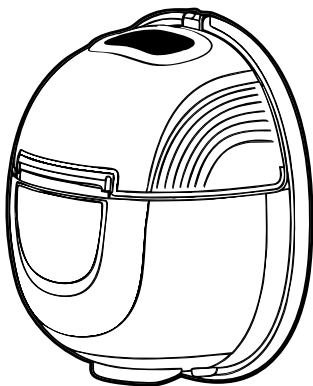
OLLY

35

PL Instrukcja obsługi
EN User manual
SK Používateľská príručka
CZ Návod k obsluze
RO Instrucțiuni de utilizare
DE Bedienungsanleitung
BG Инструкция за експлоатация

HU Használati útmutató
LT Naudojimo instrukcija
FR Notice d'utilisation
UK Посібник користувача
ES Instrucciones de uso
PT Instruções de utilização

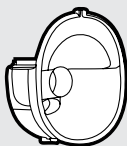
A X1

B X1

C X1

D X1

E X1

F1 X1

19mm

F2 X1

21mm

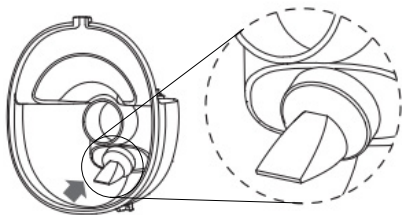
G X1


H X1


2.1

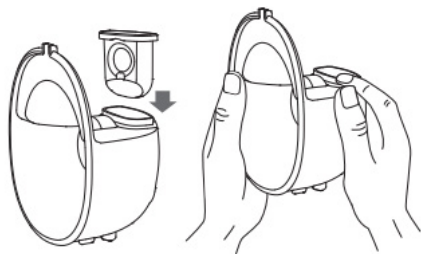
2.2

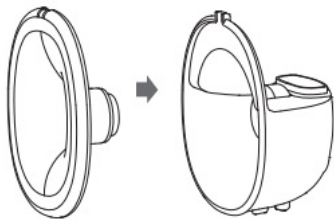
2.3

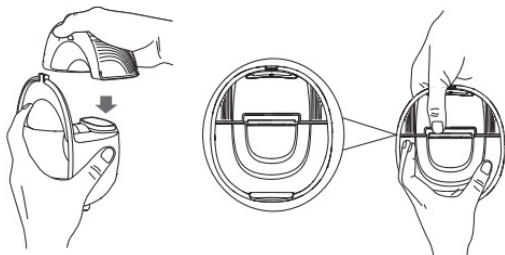
2.4

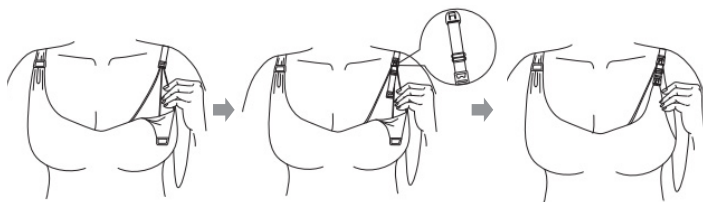
4.

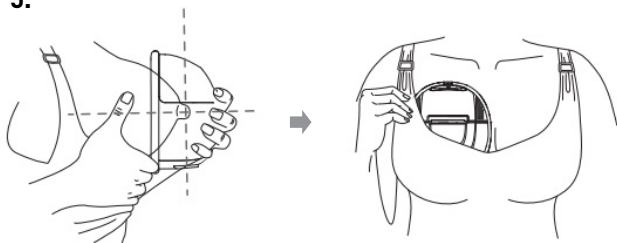
5.

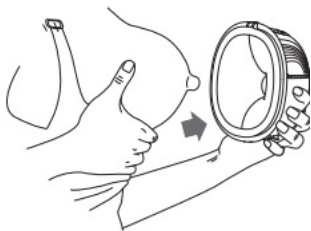
6.1

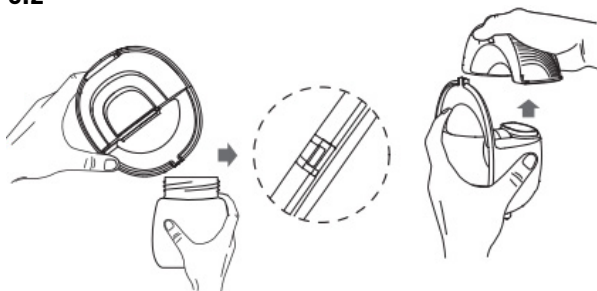6.2

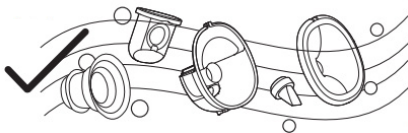6.3

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Before using the product, read the original instruction manual, follow its instructions and keep it for future reference. Pay particular attention to the safety recommendations.

PRECAUTIONS

1. Please read the instruction manual carefully before use.
2. Fully charge the product before first use. Do not overcharge the battery. Lithium batteries can only be replaced by the manufacturer, and replacement with the wrong battery model may result in a risk of explosion. Dispose of used batteries in accordance with local regulations.
3. If the product is not used for a long time, the battery will discharge. Connect it to the charger to reactivate it. Wait ten minutes until the light indicates activation. Then continue charging until fully charged. If it does not activate after more than half an hour, contact customer service.
4. The pump motor and other electrical parts **MUST NOT** be washed with water or sterilized in a steam sterilizer.
5. Please use the charging cable included with the product. Do not connect charging cables from other manufacturers to the motor to avoid damaging the device.
6. Please use a DC power supply with an AC input voltage of 100-240V and an output voltage of DC 5V, 1A.

ENVIRONMENTAL PROTECTION

The crossed-out wheeled bin symbol on the equipment or its packaging indicates that the product should not be disposed of with other waste. It is the user's responsibility to take used equipment to a designated collection point for proper treatment. Information about the available collection system for used electrical equipment can be found at the shop's information desk and at the municipal office. Appropriate handling of used equipment prevents negative consequences for the environment and human health.

	Component name	Materials	Operating temperature	Cleaning instructions
A	Motor	—	—	Wipe with a soft cloth
B	Membrane	Silicone	-20°C ~ 200°C	Rinse with water / Not suitable for long-term sterilization
C	Milk container	PP	-20°C ~ 120°C	Boiling in water / Microwave / Dishwasher / Safe for UV sterilization
D	Duckbill valve	Silicone	-20°C ~ 200°C	Rinse with water / Not suitable for long-term sterilization
E	Funnel	Silicone	-20°C ~ 200°C	
F	Funnel inserts 19/21 mm	—	-20°C ~ 200°C	
G	Charging cable	—	—	Wipe with a soft cloth
H	Bra adjustment clip	—	—	
—	Instructions	—	—	—

1. CONTROL PANEL

BUTTONS

ON/OFF	ON/OFF button	• Long press — turn on/decrease level. • Short press — pause. • After 10 minutes of pause, the breast pump will automatically turn off.
M	Mode switch button	The default mode M starts when the device is turned on. Thanks to the built-in memory function, the breast pump always starts in the last used mode.
— +	Suction strength adjustment button	All modes have levels from 1 to 9 to choose from.

DISPLAY INSTRUCTIONS

	Battery indicator	A flashing battery symbol indicates that the device should be charged as soon as possible. When the battery level is too low, the device will automatically switch off. During charging, the symbols light up one after the other. When all three symbols are lit continuously, charging is complete and the battery is fully charged.
	Time display	The device will automatically turn off after 30 minutes of continuous operation.
	Level display	Levels from 1 to 9
	Massage mode	Gentle stimulation to induce milk flow.
	Stimulation mode	Variable frequencies to get more milk.
	Suction mode	Simulate natural suction of baby
	Expressing mode	Slow and deep suction to completely empty the breast.

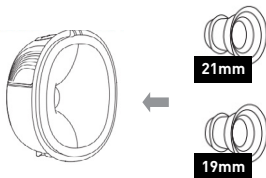
2. INSTALLATION

NOTE: Please sterilize all parts of the breast pump and wash your hands before assembly.

1. Slide the duckbill valve onto the milk collection container and make sure it is securely attached.
2. Insert the silicone membrane into the milk collection container and press it down with your hands to ensure that the edges fit snugly, providing a tight seal.
3. Attach the funnel to the milk container, making sure that the drainage groove on the funnel is aligned with the groove on the container that prevents milk from flowing back. Then press down on the edges with both hands to ensure a tight connection.
4. Attach the motor part to the milk container, align the cutout under the motor with the silicone membrane, and slide it on from the top to fit snugly against the container, ensuring a complete seal.

Measure the size of your nipple and select the funnel size that is best for you.

The set includes funnel inserts in sizes 19 mm and 21 mm. The original breast pump body is 24 mm in size.

**Do not exceed the maximum capacity of the milk container.**

Sit comfortably in a chair, lean forward slightly (you can use a pillow to support your back). Make sure the breast pump and funnel are positioned vertically.

Sitting posture

3. HOW TO USE

1. Make sure that all parts of the breast pump have been sterilized and are correctly assembled.
2. Wash your hands and massage your breasts with a warm, damp towel before use.
3. Press the funnel firmly against your breast so that there are no gaps and make sure that the nipples are centered.
4. Hold the breast pump upright. Be careful not to move the device or your breast, as this will prevent the breast pump from sucking.
5. Hold down the power button to start. Select a suction strength that is comfortable for you.

NOTE: The breast pump will automatically turn off after 30 minutes of operation.

4. INSTRUCTIONS FOR ADJUSTING THE BRA CLIP

Adjust the length of the bra straps for greater comfort while expressing milk. Hook one end of the bra clip onto the inner clip of the nursing bra. Securely fasten the other end of the clip to the bra strap.

5. STARTING TO EXPRESS MILK

Place the breast shields firmly against your breasts so that there are no gaps and make sure that your nipples are centered. Hold the breast pump upright. Be careful not to move the device or your breasts, as this will prevent the breast pump from suctioning properly.

6. ENDING THE MILK EXPRESSION SESSION

When you have finished expressing milk, remove the breast pump from your breast. Hold the breast pump upright, disconnect the motor part and hold the milk container to pour the milk into the bottle. Make sure that the groove on the container is aligned with the opening of the bottle to prevent spillage.

1. When you have finished expressing milk, hold down the power button for a few seconds to turn off the device, then gently break the seal between your breast and the funnel with your finger, and carefully remove the breast pump from your breast.
2. Separate the milk container from the motor. Align the groove on the container with the bottle and pour the milk into a feeding or storage bottle.
3. Wash all other parts used, except for the electric motor, and then disinfect them.

7. CHARGING

When charging, connect the adapter to a power outlet and make sure it is securely plugged in. The motor should be turned off during charging. The motor will not charge if it is turned on. The indicator light will flash during charging. When fully charged, all indicator lights will be lit.

NOTE: The breast pump will not work while charging.

8. TROUBLESHOOTING

TOO LITTLE OR NO SUCTION

- A. Check that all parts are correctly assembled.
- B. Check that the center of the funnel is positioned on the nipple and fits snugly against the breast, making sure no air can enter.

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