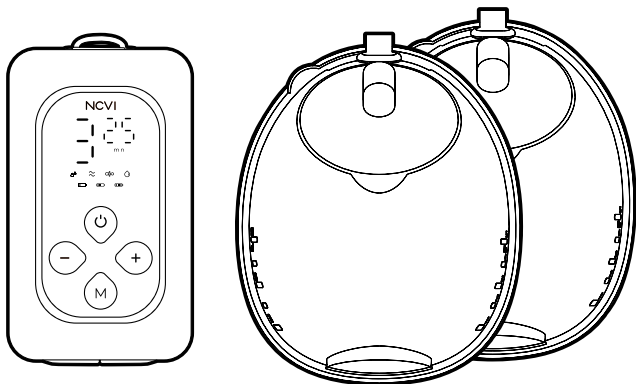
OLLY 45

PL Instrukcja obsługi
EN User manual
SK Používateľská príručka
CZ Návod k obsluze
RO Instrucțiuni de utilizare
DE Bedienungsanleitung
BG Инструкция за експлоатация

HU Használati útmutató
LT Naudojimo instrukcija
FR Notice d'utilisation
UK Посібник користувача
ES Instrucciones de uso
PT Instruções de utilização

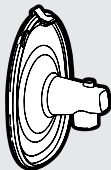
A X1

B X1 21/24mm

C X1


D X1

E X1

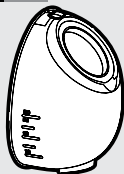F X1


G X1


H X1

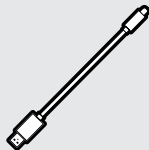
I X1

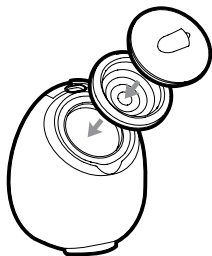2.1

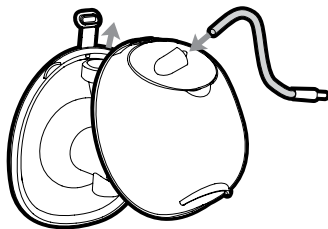2.2/2.3

4.

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Before using the product, read the original instruction manual, follow its instructions and keep it for future reference. Pay particular attention to the safety recommendations.

PRECAUTIONS

1. Please read the user manual carefully before use.
2. Charge the device completely before first use. Do not charge the battery for too long. Lithium batteries may only be replaced by the manufacturer. Replacing with the wrong battery model may result in explosion. Dispose of used batteries in accordance with local regulations.
3. If the device has not been used for a long time, the battery may become discharged. Connect it to the power supply to reactivate it. After connecting, wait about 10 minutes until the screen lights up, which indicates activation. Then continue charging until fully charged. If the device does not activate after more than 30 minutes, contact customer service.
4. The pump motor and other electrical parts **MUST NOT** be washed in water or sterilized in a steam sterilizer.
5. Please use the charging cable included with the product. Do not connect a cable from another manufacturer to the motor to avoid damaging the device.
6. Please use a power adapter with an input voltage of AC 100~240V and an output voltage of DC 5V, 2A.

ENVIRONMENTAL PROTECTION

The crossed-out wheeled bin symbol on the equipment or its packaging indicates that the product should not be disposed of with other waste. It is the user's responsibility to take used equipment to a designated collection point for proper treatment. Information about the available collection system for used electrical equipment can be found at the shop's information desk and at the municipal office. Appropriate handling of used equipment prevents negative consequences for the environment and human health.

	Component name	Materials	Operating temperature	Cleaning instructions
A	Engine	—	—	Wipe with a soft cloth
B	Funnel inserts 21/24 mm	Silicone	-20°C ~ 200°C	Rinse with water / Not suitable for long-term sterilization
C	Funnel (28mm)	PP	-20°C ~ 120°C	Boiling in boiling water / Micro-wave / Dishwasher / Safe for UV sterilization
		PS	-20°C ~ 149°C	
D	Membrane	Silicone	-20°C ~ 200°C	Rinse with water / Not suitable for long-term sterilization
E	Duckbill valve	Silicone	-20°C ~ 200°C	
F	Milk storage container (150 ml)	PP	-20°C ~ 120°C	Boiling in boiling water / Micro-wave / Dishwasher / Safe for UV sterilization
		PS	-20°C ~ 149°C	
G	Cover	PP	-20°C ~ 120°C	Rinse with water / Must dry out before using
H	Tube	—		Wipe with a soft cloth
I	USB cable	—		

1. CONTROL PANEL

BUTTONS

ON/OFF	Power button (ON/OFF)	- Long press — turn on/decrease level. - Short press — pause. - After 10 minutes of pause, the breast pump will automatically turn off.
M	Mode switch button	The default mode M starts when the device is turned on. Thanks to the built-in memory function, the breast pump always starts in the last used mode.
— +	Suction strength adjustment button	All modes have levels from 1 to 9 to choose from.

DISPLAY INSTRUCTIONS

	Battery indicator	A flashing battery symbol indicates that the device should be charged as soon as possible. When the battery level is too low, the device will automatically switch off. During charging, the symbols light up one after the other. When all three symbols are lit continuously, charging is complete and the battery is fully charged.
23	Time display	The device will automatically turn off after 30 minutes of continuous operation.
5	Level display	Levels from 1 to 9
	Massage mode	Gentle stimulation to induce milk flow.
	Stimulation mode	Variable frequencies to get more milk.
	Suction mode	Simulate natural suction of baby
	Expressing mode	Slow and deep suction to completely empty the breast.

2. HOW TO ASSEMBLE THE DEVICE

1. Insert the membrane into the milk reservoir, making sure there are no leaks.
2. Insert the silicone duckbill valve into the breast shield.
3. Snap the funnel and milk collection container together at the top, then slide the container onto the funnel.

Before use, make sure your hands and all parts of the breast pump are clean and dry.

3. HOW TO USE

The breast pump uses a two-phase suction mode that mimics the natural rhythm of a baby's suckling. The device offers four different modes and nine suction levels. The suction intensity increases with each level.

1. Make sure that all parts of the breast pump have been sterilized and are correctly assembled.
2. Wash your hands and massage your breasts with a warm, damp towel before use.
3. Press the funnel firmly against your breast, leaving no gaps, and make sure that the nipples are centered.
4. Hold the device upright. Be careful not to change the position of the product or your breast, as this may cause the suction to stop.
5. Hold down the power button to start. Select a suction strength that is comfortable for you.

TIPS

We recommend that mothers try different modes and suction levels to suit their individual breastfeeding situation.

Remember to clean and dry the remaining parts of the breast pump (funnels, duckbill valves, membranes) to avoid milk stains. If you do not plan to feed your baby immediately, store the milk in the refrigerator.

4. TROUBLESHOOTING

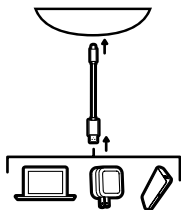
TOO LITTLE OR NO SUCTION

- A. Check that the duckbill valve and membrane are correctly installed.
- B. Check that the funnel at the outlet (the so-called bell) is correctly seated.
- C. Check that the connection between the air hose and the main unit is correctly installed.
- D. Check that the air hose is not twisted, pinched or loose at both ends at the main connector.

CAUTION: Maintain the correct posture when expressing milk. Please do not lie on your side to prevent milk from flowing back.

5. CHARGING

The breast pump should be turned off during charging. It will not charge if it is turned on. Plug the adapter into a power outlet. The battery symbol will flash during charging. When the battery symbol is steady, the battery is fully charged.

CONNECTING THE CHARGING CABLE:

Connect the USB cable to the USB port on the pump motor. The breast pump includes a USB cable and can be charged using:

1. USB ports on a compute (output: DC 5V/2A);
2. An AC power adapter (input: AC 100-240V / 50-60Hz; output: DC 5V/2A);
3. USB power bank (output: DC 5V/2A).

OPERATING CONDITIONS

Operating temperature range: **5°C do 40°C**

Relative humidity range: **20% ~ 80%**

Atmospheric pressure range: **850 hPa ~ 1060 hPa**

Long-term storage should be in a dry and cool place.

6. POWER SUPPLY

ADAPTER

Plug the adapter into the power outlet, hold down the power button to turn on the device, select the mode and suction level. A short press of the power button turns the device off.

LITHIUM BATTERY

The product is equipped with a lithium battery. During use, press and hold the power button to turn on the device, select the mode and suction level. Press and hold the power button to turn off the device. When the battery symbol flashes, it means that the energy level is low — recharge the device as soon as possible.

CAUTION

Keep the membrane, tube connector, tube adapter, air tube, and other components dry to prevent damage to the motor by water intake!

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